# i-See Speech

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**Read this manual carefully before using the i-See Speech.  
Keep the user manual for future reference.**

## Table of contents

EN

|    |                                   |    |
|----|-----------------------------------|----|
| 1  | Foreword.....                     | 27 |
| 2  | Safety .....                      | 27 |
| 3  | Device overview.....              | 28 |
|    | Contents of the box.....          | 28 |
|    | Controls .....                    | 28 |
|    | Connections.....                  | 29 |
|    | Installation.....                 | 29 |
| 4  | Use .....                         | 30 |
|    | Turning the device on or off..... | 30 |
|    | Switching between modes .....     | 30 |
| 5  | Live view mode.....               | 30 |
|    | Controls.....                     | 30 |
|    | External input.....               | 32 |
| 6  | Live view settings.....           | 32 |
| 7  | i-See Speech Mode.....            | 34 |
|    | Switching to speech .....         | 34 |
|    | Start reading .....               | 34 |
|    | Image view or digital view.....   | 35 |
|    | Zone detection.....               | 35 |
|    | Autocue.....                      | 36 |
|    | Languages.....                    | 36 |
|    | User interface.....               | 36 |
| 8  | Aquity Speech Menu.....           | 38 |
|    | User preferences.....             | 38 |
|    | Read-aloud preferences.....       | 38 |
|    | System settings .....             | 39 |
|    | Documents.....                    | 41 |
|    | Save.....                         | 41 |
| 9  | Troubleshoot.....                 | 42 |
|    | Maintenance.....                  | 43 |
| 10 | Recycling.....                    | 43 |
| 11 | Warranty.....                     | 44 |
| 12 | Technical specifications .....    | 44 |
| 13 | Remote control.....               | 93 |
| 14 | Supplier information .....        | 94 |
| 15 | Manufacturer information.....     | 94 |

This document concerns the original general user manual,  
has been prepared by and is commissioned by Rehan Electronics Ltd.  
The Aquity, i-See and i-See Speech product names are trademarks of Rehan Electronics Ltd.

# 1 Foreword

This user manual is intended to describe this device's use, installation and maintenance. Read this manual carefully before use and observe all safety instructions to prevent damage to your health, the device or the environment.

The i-See Speech is designed to support reading for people with low vision or dyslexia. The device is intended to enlarge documents/media up to 65x, enhance contrast and read text aloud. This is possible for documents on paper and in digital form. In addition, the device can read the text in different languages and speeds.

The device is intended as a desktop for in-home or office use, with easy-to-use controls on the display panel and optional remote control. The large A3-size tableau makes it easy to move the reading line back and forth. In addition, you can connect external sources by HDMI to display on the device.

EN

## 2 Safety

- Place the i-See on a stable and horizontal surface to prevent it from falling.
- Ensure that the i-See is placed at least 10 cm / 4 inches from the edge of the table.
- Only use your i-See indoors. Do not expose the device to extreme temperature fluctuations or humidity. Do not place your i-See next to a heat source.
- Ensure that the i-See is placed in a well-ventilated room.
- Only connect the adaptor to a wall socket outlet using the supplied cables.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the i-See, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use precautions for implanted devices, like a pacemaker. Maintain at least 15cm/6 Inches between the speaker or headphone and the implanted device. The device also contains magnets in the display's upper corners and in the tableau's remote holder. See magnetic strength details in the technical information. If in doubt, contact your specialist.
- Be careful when handling the moving parts of the tableau of the screen.
- Do not use the i-See or its tableau to support walking, sitting down or standing up.
- Be aware that listening to a high sound level could result in hearing damage.
- Switch off the device if you do not intend to use it.
- The device may give possible reading errors due to external influences such as ambient light, paper type or print quality. Check critical data such as doctor prescriptions or bank details with the live view mode or with other persons.

### 3 Device Overview

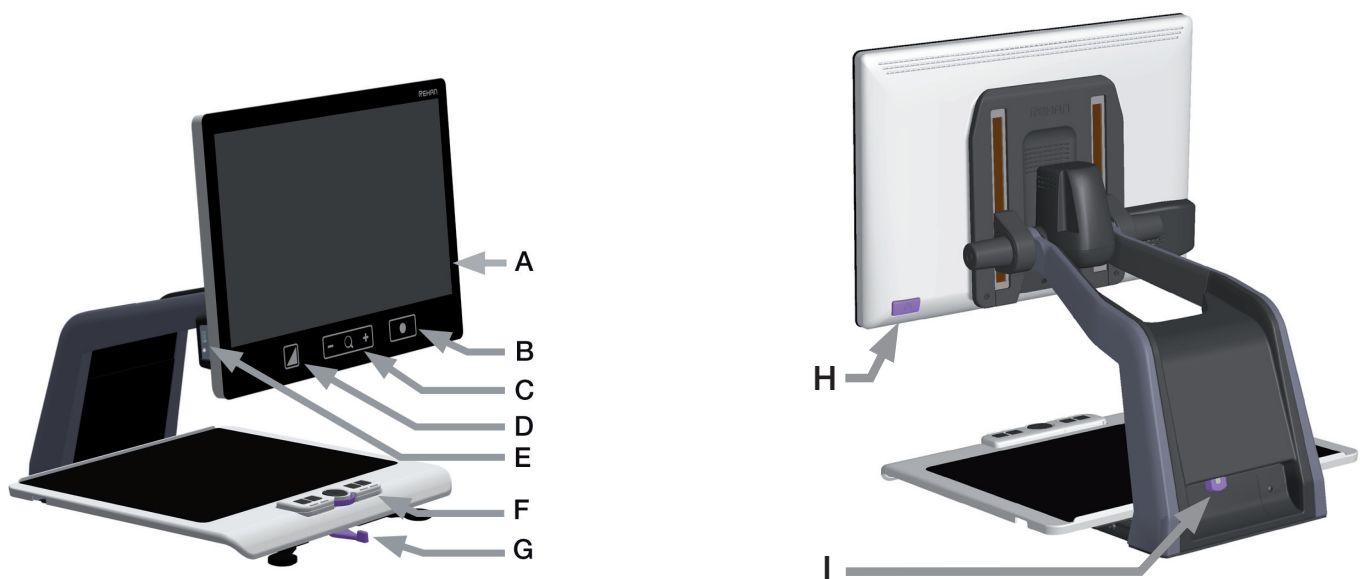
#### Contents of the box

- i-See Speech device
- Power Adaptor
- User Manual
- Headphones
- i-See training Doming Label
- HDMI Cable
- Remote Control and Remote control receiver

#### Controls

EN

The device has separate touch controls underneath the screen. You can make use of the full touch screen also. Besides this, the device can be controlled with a remote control.

- A - Full touch screen
- B - Touch button 3 (B3)
- C - Touch button 2 (B2)
- D - Touch button 1 (B1)
- E - USB, headphones connections

- F - Remote Control
- G - Brake handle
- H - ON/OFF button
- I - Connections

The remote control has buttons on both sides.

The side with two buttons is for Live view.

The side with six buttons is for Speech.

See chapter 13 for more information about the remote control.


#### Tableau

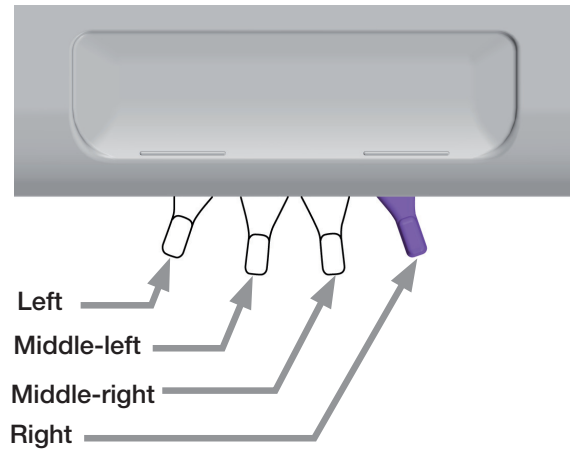The i-See Speech contains a tableau on which you can place your reading material. The tableau can move from left to right and from front to back.

#### Brake handle

In front of the tableau of the i-See Speech, you will find a single brake handle with which you can control the movement of the tableau. The handle has four positions:

- Left: The tableau can be moved freely.
- Middle-left: Free movement from left to right. 5-millimetre increments can adjust the front-to-back direction.
- Middle-right: Free movement from left to right. Front-to-back direction is blocked.
- Right: Tableau position is completely locked.


## Connections

On the left side of the display:

- Headphones
- USB

At the back of the device:

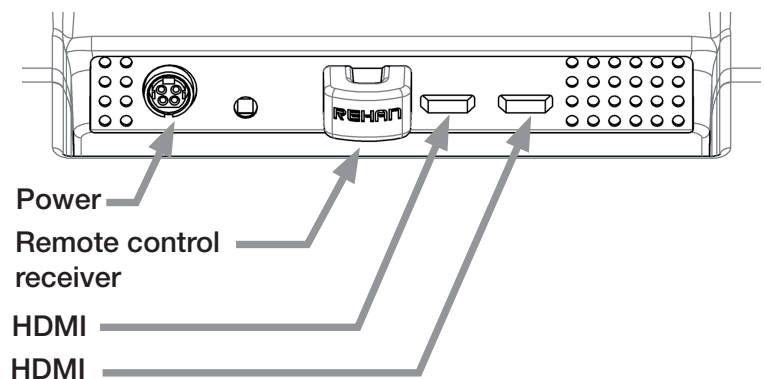- Power
- Remote control receiver
- 2x HDMI

EN

## Installation

The i-See will be delivered to you in a pre-assembled state.

### *Get the device out of the box*

The top of the box can be opened, allowing the device to be lifted out of the box. Note that the weight of the device is around 17 kg. Lift the device from the back by holding both arms. Place the i-See on a stable and horizontal surface. Remove the foam blocks and store them in the original box.

### *Positioning*

#### **Height:**

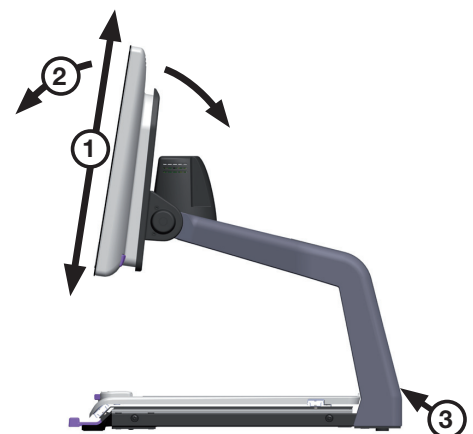
The height of the screen can be adjusted by firmly holding the screen on the left and right side, and pulling the screen up or down to your preferred position.

#### **Tilt:**

The screen can be tilted by firmly holding the screen on the left and right side and tilting the screen in your preferred position.

### *Power adapter*

Place the round connector from the power adapter to the power input at the back of the device. It will fit only in one direction so keep the flat part of the connector towards you. Connect the other end of the power adapter into the wall socket.

# 4 Use

## Turning the device on or off

The on/off switch of the i-See Speech is located at the lower right hand side on the back of the monitor (see the Controls pictures in chapter 3). To start up or shut down the device, hold it for 2 seconds until you hear the startup or shutdown tone.

During start up the logo appears on the screen and when this disappears, the live image of the camera is displayed. This is the “Live view mode” of the device. The device is now ready to use.

## Switching between modes

The i-See is divided in different modes to get the best performance for the different usage. So it is possible to set the device to these modes: Live view, Speech and external input.

EN

Press the ON/OFF button once while the device is on, to step between the different modes. Once a remote control is connected, you can flip the remote control to switch between the Live view and the Speech mode.

The external input is only available when an HDMI cable is connected to one of the HDMI ports at the back of the i-See Speech and to an external device (see the Connections picture, chapter 3.). An external device can be a camera or a computer. The device can be switched to this mode by pressing the power button until the external input icon is shown, or by pressing the external input button on the remote control.

# 5 Live view mode

When you turn on the device it will start in Live view mode. In the Live view you will see the tableau or the documents on it, which you can use for enlargement or contrast improvement.

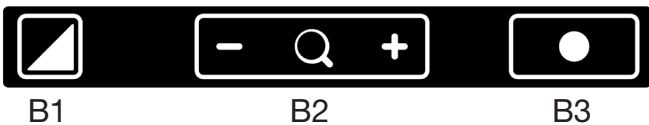
## Controls

The three buttons (B1, B2 and B3) can be found below the screen and consist of touch sensitive pads.

For improved feel of the position of these touch sensitive pads, the i-See Training Doming Label can be pasted over these touch sensitive pads.

The functions of the three touch sensitive pads are as follows:


## Display modes

Tap this icon (B1) to go to the next available display mode.

Press and hold this icon to quickly switch between your last used high contrast mode and photo mode.

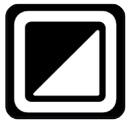Swipe over this icon from left to right or right to left to go to the next or previous available display mode. “Photo mode” is the icon with multiple colors (A).

B1

## Zoom

Tap or press and hold the minus or plus icon to zoom out or in (B2). The magnification level will be shown on the screen.

B2

## Block auto focus

Tap the magnifier icon to refocus on the reading material and block the auto focus on that level. Tap a zoom button to unblock the auto focus.

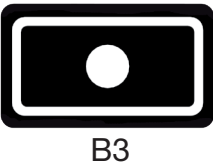
**LED Lightning**

The brightness of the LED lighting on the tableau can be adjusted to prevent reflections or to be less intense in dark environments by reducing the light from the LEDs or turning them off. To do this, press and hold the magnifying glass icon until a light bulb icon appears on the screen with a value between 0 and 5 (0 indicates the LEDs are off and 5 indicates the LEDs are at the brightest level). Use the minus and plus touchpad icons to change the value down or up as required. *Please note that reducing the brightness may result in low image quality, increased sensitivity to shadows or cause text to disappear. If this happens, increase the brightness or adjust the contrast accordingly.*


**Brightness**

By tapping this icon (B3) you can cycle through 5 brightness levels. *Brightness can't be adjusted when reading aids are in use.*

**Contrast**

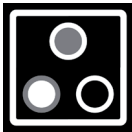In contrast mode, swipe over this icon (B3) from left to right (and hold your finger on the right side of the rectangle) or right to left (and hold your finger on the left side of the rectangle) to gradually increase or decrease the contrast. The contrast icon is shown in grayscale.

*Contrast can't be adjusted when reading aids are in use and can only be adjusted in high contrast modes, not in photo modes.*

**White balance**

In photo mode, swipe over this icon (B3) from left to right (and hold your finger on the right side of the rounded rectangle) or right to left (and hold your finger on the left side of the rounded rectangle) to gradually change the white balance to the colder, more blue side or change the white balance to the warmer, more orange side. The white balance icon has multiple colors.

*White balance can't be adjusted when reading aids are in use and can only be adjusted in photo modes, not in high contrast modes.*

*White balance adjustment is turned off by default. To turn it on read Chapter 6 Live view settings.*

**Advanced reading aids**

Press and hold the brightness icon (B3) to enter the Advanced reading aids menu.

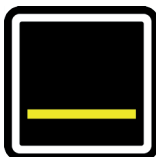
The following reading aids are available:

horizontal  
blanking

vertical  
blanking

horizontal line

vertical line

Swipe up or down over this icon to select the desired reading aid required. Each reading aid has 5 settings that are indicated by a number shown in the icon.

Once the desired reading aid is selected, tap this icon to set the desired setting of the reading aid. To change the setting value you can tap this icon to go to the next level or you can swipe from left to right.

To change to a different reading aid then swipe up or down over this icon and then tap to set the desired level of the reading aid selected.

To exit from the advanced reading aids menu press and hold this icon (B3) for 2 seconds. The reading aid will now disappear from the screen.

*Brightness, contrast and white balance can't be adjusted when reading aids are in use.*

## External input

It is possible to connect an external input such as a camera or a computer. Press the on/off button once to toggle between the available inputs until the image of the external input is displayed on the screen of the i-See.

Adjust the display modes, brightness, contrast, white balance and advanced reading aides of the external image as described above.

Tap or press and hold the minus or plus icon to zoom out or in. When zoomed in, it is possible to pan through the image by pressing and holding the magnifier icon and sliding your finger in the direction you want to pan the image. Always start from the magnifier icon to start panning.


## 6 Live view settings

EN

The i-See Speech contains a settings menu with which you can (de)activate certain functions of the Live view mode. You can access the settings menu by ensuring the device is switched off, then turning the device on using the on/off button; press and hold until you hear a beep after the start up tone.


By tapping the brightness icon (B3) you can cycle through multiple screens of selectable options. Tap the plus icon of the zoom button (B2) to highlight which mode you wish to change.

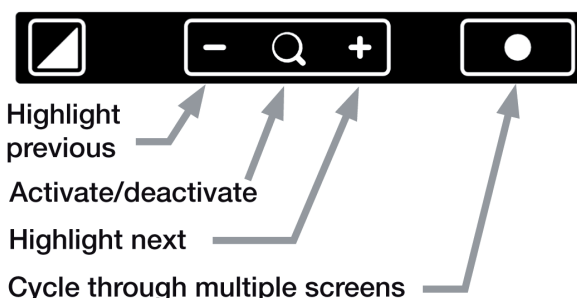You can recognize the highlighted function by the white rounded rectangle around it. Tap the magnifier icon to activate/deactivate your selection.


The alternative method is by using the remote control. The wheel is used for highlighting the function. The button on the right ('external input') to cycle through multiple screens. With the button on the left you can activate/deactivate your selection.

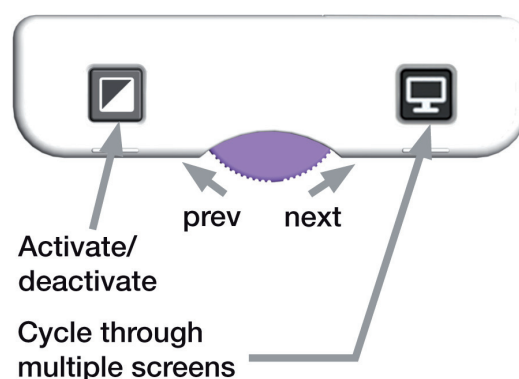
The process of saving the changes and exiting is described at the end of the chapter.

### Display modes

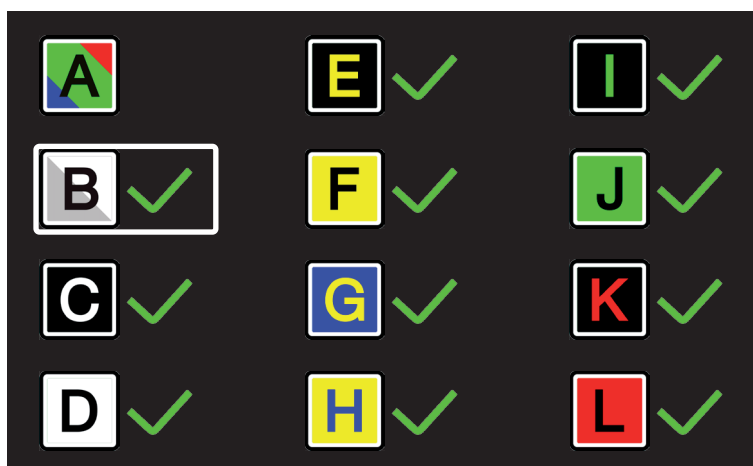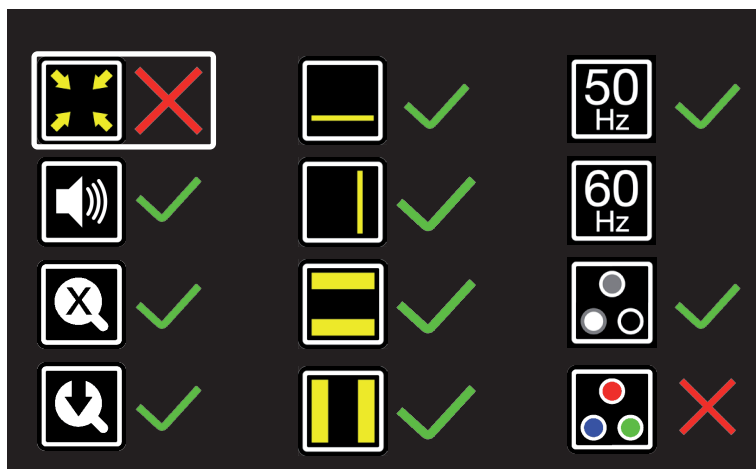On the first screen you can select whether you want the photo mode to have realistic or high saturation colors by activating the photo mode (A) and select which high contrast mode you want available in your Live view mode by activating the high contrast mode you want activated/deactivated.


The second screen shows advanced functions:

### Crop zoom

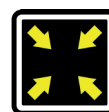The crop zoom function crops the image on screen to 50% so that a zoom level of 1x can be achieved.

*This function is turned off by default.*

### Sound feedback

When this function is on you hear a beep when you change a function. *The function is turned on by default.*

### Magnification indicator

When you zoom in or out a magnification indicator shows up in the top left corner that shows what the magnification factor is. *This function is turned on by default.*

### Zoom preset

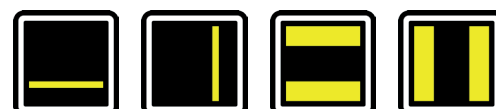
When this function is on the device will go to the last used magnification in high contrast modes and photo mode when you switch to these functions. *This function is turned on by default.*

### Advanced reading aids

This will let you turn reading aids on or off individually.

*This function is turned on by default.*

### Refresh rate / frequency

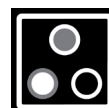
Depending on your location the mains power frequency will be 50 or 60 Hertz. By selecting the right setting you will suppress flickering caused by surrounding light. *This function is set to 50Hz default.*

### Contrast

With this function you can turn the manual contrast adjustment on or off.

*This function is turned on by default.*

### White balance

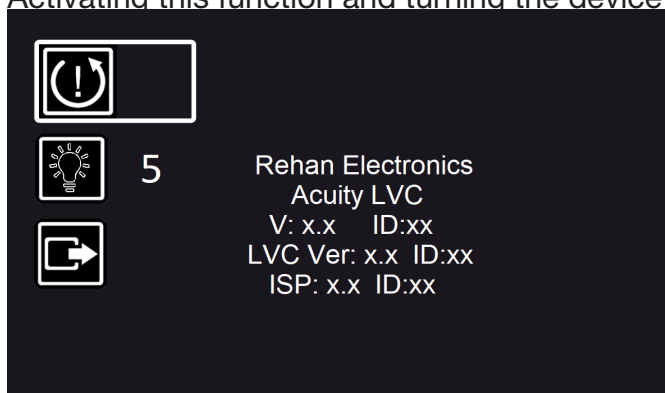
Select whether you want the ability to manually set the white balance in the photo modes. *This function is turned off by default.*

On the third and last page of the settings you will find the factory reset, save and exit and the installed software version.

### Factory reset

Activating this function and turning the device off and on again will reset all advanced functions and last used magnifications to their default setting. *The factory reset function is a permanent action and can not be undone.*

### LED Lightning

The brightness of the LEDs can be adjusted between values of 0 and 5 (0 indicates the LEDs are off and 5 indicates the LEDs are at the brightest level). *Please note that reducing the brightness may result in low image quality,*

increased sensitivity to shadows or cause text to disappear. If this happens, increase the brightness or adjust the contrast accordingly.

**Save and exit**

Activating this function saves any changes and exits the menu. If you didn't make changes, the device stays on and can be used immediately. In case you made changes, the device will automatically go through a power cycle (turn off and on again) and the changes are saved.

**7 i-See Speech mode**

The i-See speech has the ability to read documents aloud. Therefore, the device switches to a different camera to fully frame an A4 or US letter document.

EN

**Switching to Speech**

When the i-See is just turned on, an hourglass icon appears in the top right corner of the screen. This indicates that the text recognition software is not completely started. When the hourglass has disappeared, the speech software is ready, and the device can be switched to speech mode by pressing the power button shortly or by flipping the remote control when it is connected.

Switching back to Live view mode can also be done by shortly pressing the power button or by flipping the remote control when it is connected. If you switch while reading, the reading will pause, and the position will be remembered, so when you switch back to speech, you can continue reading where you left off.

**Start reading**

When the device is switched to the Speech mode, it shows the Start screen. On this screen there's a full page live view of the tableau. The yellow marker indicates the scan area.

Place the document on the tableau so the desired text is within the yellow area. Feel free to rotate the document 90 degrees in case the document is printed in landscape. The software will automatically correct the position of the text, even when it is upside down.

Press the play button in the bottom right corner of the screen. Or press inside the yellow marker on the screen to capture the document. The device will now analyse the text of the document. During this analysis, the device gives a tick-tock sound to indicate it is in progress. This process can be interrupted by pressing the stop button in the bottom right corner of the screen.

**Recognition quality may vary due to ambient light, paper and print quality. Do not rely entirely on processed text alone. Check critical documents (i.e. doctor prescriptions) in image view or in the Live view mode.**


When recognition is completed, the device will display the document in the desired text form. By default, the document is displayed in "image view". A button interface bar is displayed at the bottom of the screen to control the device. There you can start/pause reading or do other functions. See the section "Reading functions".

Press the stop button in the bottom right corner to stop reading the document. The device then returns to the start screen.

**Image view or digital view**

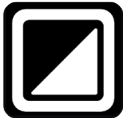The i-See Speech has the ability to display the recognised text as an image or as digital text. With the image view, the layout of the document is still visible, which can be useful for magazines or newspapers. When choosing digital text, only the text is displayed, which makes reading along more comfortable. To be able to switch between the views, the recognition must be completed first (play button).

By default, the document will be displayed in “image view”. To switch between the image or digital view, hold the “contrast button” (B1) underneath the screen pressed for two seconds. This is an easy method to switch between different views.

In both “views”, changing the text/background colours to create maximum contrast is possible. To do this, tap the contrast button briefly to step through the different colour scenes. Swipe over this icon from left to right or right to left to go to the next or previous available colour scenes.

**Digital View options**

For the digital view, there are some options for how to display the text. Therefore, we have to open the menu. Press the menu icon at the screen’s bottom left corner to open or close the menu. The menu always opens in “user preferences”. Under the item: “View”, you can specify which view you start in by default: image or digital. When this setting changes, it will be immediately active in the current situation. Also, the following captured documents will be displayed in this setting.

The digital text can be displayed in 3 options: Full text, a reading line or per word. Select the preferred option, and the digital text will always be displayed in this method when digital text is switched on. The list of colour scenes can be adjusted as well. See section: Settings.

**Zone detection**


In the image view, the layout of the document is still visible. This can be useful for magazines or newspapers, because several articles can be on one page or in multiple columns. In this case, there is a chance that the software will confuse them or read them in the wrong order. For this, you can turn on Zone detection.

Press the menu icon in the bottom left corner of the screen to open or close the menu. The menu always opens in “user preferences”.

Under the item: “Zone”, you can choose between “zones” or “free mode”.

With “zone mode”, each text block is detected as its own zone. By tapping on these zones, you can mark which pieces of text you want to read. Then, press the play button to start reading. The device will follow the order you have selected. This is also displayed with a number at the center of each selected zone.

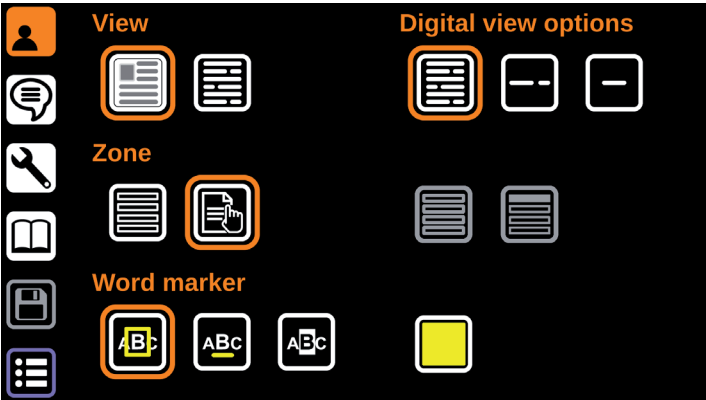In free mode, the device starts reading from top to bottom.

**Zones in digital text**

You can only select zones in the image view. The selected text zones can be read in digital view by pressing the play button for two seconds. Then, the device only switches the selected zones to the digital view. Once the selection is finished, the device switches automatically back to the zone selection screen. Once the document contains pictures or other elements, it would be possible in the zone selection screen to tap on the picture or element to display it enlarged. Then, tap again to close it.

Autocue

In digital text view, the device has the functionality for autocue. The text slowly scrolls on the screen without a word marker or reading voice within this function. Depending on the digital view options, the text will be displayed as full text, which scrolls down up or as a line, which scrolls left to right.

The autocue can be started by double-tapping the touchscreen. It will automatically start scrolling and can be paused by the play/pause button in the bar. Further, only the speed can be changed with the speed icon on the button bar.

To go back to read the document with reading voice, double tap on the screen again. The document will be closed when pressing the stop button, and the device will go to the start screen. *Autocue is not available when the device is set to Interface preset 3.*

EN

Languages

The i-See speech can detect the language of the document. This way, the device uses the correct voice to pronounce the text correctly.

The device has 5 preferred languages for this purpose. If the detected language is not among them, it uses the system language, the same language as the menu.

You can choose these 5 preferred languages. Press the menu icon at the bottom left of the screen. And go to the second item: “Read aloud preferences”, the text balloon icon. There you can select 5 languages. If you press on a language, you will get a list of available languages to choose from. This way, you can make a selection of the languages you expect to read documents from.

If the device detects a wrong language, you can uncheck a language by pressing the checkmark in front of the language. Now this language is not used during detection. Or, if you want to have a document forcibly read out in a specific language, mark only the desired language.

If you change the language while reading, you need to rescan the document again to let the device function properly.

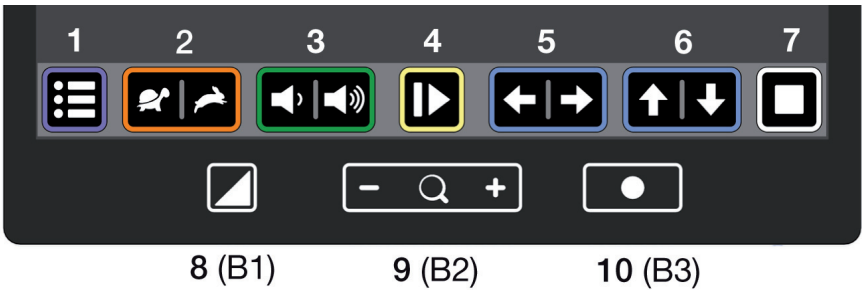
Voices

You can choose the voice you like to sound best for each language. This could be male or female or certain dialects. These voices are expressed in persons’ names. Behind the name is a play button to hear an example of the voice.

To change a voice, press on the name, and it will display all available voices related to the language. The flag before the name indicates which country/dialect it comes from.

User Interface

While reading, you can easily navigate through the text or change the reading speed with the buttons at the interface bar at the bottom of the screen.

By default, this button bar is always visible. Further, the three

buttons underneath the LCD screen, which are used for the live view mode of the device, have similar functionality for the speech mode. The device also contains a full touchscreen. The buttons each have their own function to control the reading features.

1. **Menu:** open or close the menu.
2. **Reading speed:** tap the left side to decrease the reading speed or the right side to increase the speed.
3. **Volume:** tap the left side to decrease the volume or the right side to increase the volume.
4. **Read/pause:** start or pause reading.
5. **Previous/next word:** tap the arrow left to go to the previous word or right to go to the next word. After tapping this button, the reading pauses. Holding the button pressed for two seconds skips to the previous or next paragraph.
6. **Previous/next sentence:** tap the arrow up to go to the previous sentence or down to go to the next sentence. Holding the button pressed for two seconds skips to the document's beginning or end. Or in case a document has multiple pages, to the previous or next page.
7. **Stop:** Stop reading this document and go back to the Start screen to capture another document.
8. **Contrast button** (B1): The contrast button changes the text/background colours to create maximum contrast as possible. Tap this button to step through the different colour scenes. Or swipe over this icon from left to right or right to left to go to the next or previous colour scene. Press the button for two seconds to switch between image or digital view.
9. **Zoom button** (B2): Zoom in or out to enlarge the image or the text size in digital text. Pressing on the magnifier glass at the centre will set the magnification to make the complete image visible. Or in the case of digital text, it will reset the text size to the preferred size. The preferred text size can be set in the settings. See section: Settings.
10. **Word marker** (B3): The word marker indicates the word currently being spoken. Press this button for two seconds to turn the word marker on or off. The colour of the word marker can be changed in the Settings.

The device contains a full touchscreen, allowing you to press on the corresponding word to start reading from there.

Pressing anywhere on the touchscreen will always pause reading. When paused, it is possible to scroll the text if there is more text than fits on the screen. Hold your finger on the screen and slide it in your desired direction.

### Interface presets

Three options are available to make the user interface work best with your usage. This is configurable in the menu as preset. Preset 1 is active as default, this is described as above.

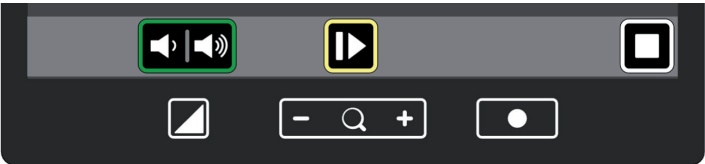
#### Preset 2: auto-hide interface

In preset 2, all the functionalities are the same as in preset 1. However, the button interface bar limits the screen size for displaying the text. Therefore, in preset 2, the button interface bar will automatically disappear after 5 seconds. This allows you to make optimal use of the screen size. Press the lower left corner to let the interface bar reappear.

#### Preset 3: simplified interface

Preset 3 is a simplified interface where only the basic features are available. Here, the device is as simple as possible and minimises the user's burden with more elaborate functions.

Only the volume buttons, play/pause and the stop button are available in the button bar. The three touch buttons underneath the LCD screen and the touchscreen can still be

used as normal.

With Preset 3, the device will always display the scanned documents in the digital view and start immediately with reading. It is not possible to switch back to the image view. Ensure the Digital View options are set to the desired layout: text block, line or word.

Also, the menu icon is hidden to prevent accidental modification of device settings. To get into the menu, press in the bottom left corner for 8 seconds, and a pop-up will appear whether you actually want to open the menu. Press OK to open the menu.

## 8 i-See Speech Menu

EN

The menu of the i-See Speech is a place where the device can be configured, but it also gives the ability to save documents and open saved documents. To open or close the menu, press the menu icon in the screen's bottom left corner.

On the left side of the menu screen, different icons will appear for these different items. Clicking on an icon will give you all the options or settings related to that category. Here we will describe the items in more detail, from top to bottom.

### User preferences

The user preferences give the options related to displaying the document.

#### *View*

Here you can specify whether the document should be displayed in image view or digital view. This will be active immediately and displayed in this mode the next time a scan is made.

#### *Digital options*

The digital text can be displayed in 3 options: full text, a reading line or per word. Select the preferred option, and the digital text will always be displayed in this method when digital text is switched on.

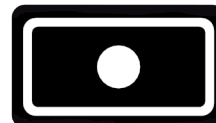
#### *Zone*

The option is to mark the document's text into zones, which can be used to set the reading order or have only small sections read.

When enabled, the zone colour of the text zones can be set to the preferred colour. And also, the preferred colour can be set when a zone is selected. Press on the icon to step through the colour options.

#### *Word marker*

The word marker indicates where the device is with reading. The current word is then marked. Here you can select whether the word is to be framed, underlined or inverted. When framed or underlined, you can specify which colour this should be. Press the colour box to step through the colours.

The word marker can be turned off during reading by pressing and holding the right touch button (B3) underneath the screen for 2 seconds.

### Read-aloud preferences

The read-aloud preferences give the options regarding the reading voice, such as language, reading speed or volume.

#### *Language*

Select 5 preferred languages the device should be able to detect. Press on the language to change it to another language. If a language should not be recognised, you can uncheck it by pressing the checkmark in front of the language.

### ***Voices***

You can choose the voice you like to sound best for each language. This could be male or female or certain dialects. Press on the voice's name to change it. Behind the name is a play button to hear an example of the voice.

### ***Speed***

Set the speed at which the document is reading. This can be set from 50 to 200%. 100% is the normal speed. This value changes when you use the speed button in the user interface while reading.

### ***Volume***

Set the volume of the reading voice.

Use the play button behind the voice to hear a preview of the speed or volume.

## **System settings**

The system settings allow you to adjust device settings such as menu language, interface preset or software update. This item consists of two pages, in the bottom right corner is an arrow to switch between the pages.

### ***System language***

The system language defines the text language of the menu. Press on the language to change it to another. This does not affect the reading of documents.

### ***System Voice***

The system voice is what you hear in the menu when system audio feedback is on. Press on the name to change it to another. Behind the name is a play button to hear an example of the voice. This does not affect the reading of documents.

### ***System Volume***

The system volume can be set separately from the read-aloud volume for when you want the system feedback to be a different volume than the actual read-aloud voice.

### ***Equalizer***

The equaliser setting can be set to change the tone of the device to have either more bass, the left option, or more treble, the option on the right (the equaliser is set to mid, the middle option, by default).

### ***Reading size***

The preferred text size to set how big the text is displayed in the digital view. Each new document will start at this text size. Or press on the magnifier glass underneath the screen to reset the text size back to this preferred text size.

### ***Audio feedback***

A setting of the device should provide feedback when a button is used. There are 3 options for this;

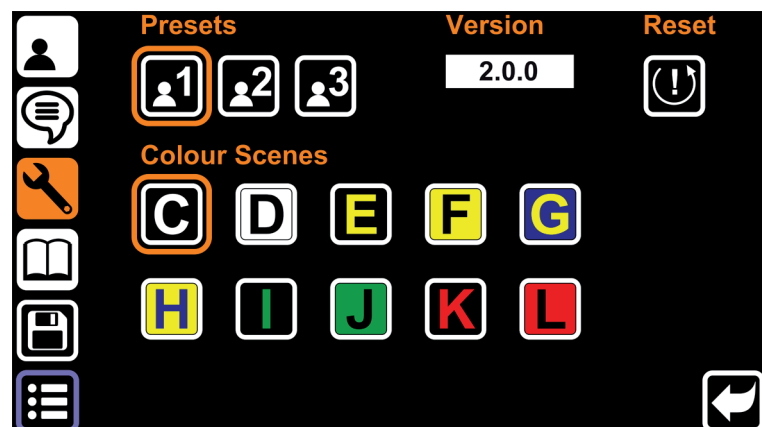
- Voice feedback: the device pronounces the function of the pressed button.
- Tone feedback: press a button, and you will hear a single beep.
- No feedback: you will hear no sound from the device except for the read-aloud voice.

## Preset – page 2

Switch between the different user interface presets to suit your use of the device best. See the section user interface for more information.

## Colour scenes – page 2

The available colours to change the text/background colours to create maximum contrast. By default, all the colours are available. Press on a colour to enable or disable the colour.

This allows you to reduce the number of colour options when using the contrast button below the screen (B1) during reading. This is only for speech mode. To set the colours in Live view mode, see section Live view settings (chapter 6).

### Version

Software version indication for service purposes.

## Reset and user preset

Restore the device settings. By default, pressing this button will reset the settings to factory defaults. But there is also the option to save your own settings to restore them quickly.

To do this, hold and press the button for 5 seconds to open a dialogue. There you can set how this button should function.

- Option 1: **Factory reset.** The next time a reset button is pressed, the device will reset everything to factory default.
- Option 2: **User preset:** With this, it saves the settings as they are currently set so that on the next reset, they are restored as it is currently set.

## Backup to USB

When a USB stick is connected to a USB port at the left side display, it can create a backup of the device. This can be useful if multiple devices need to be configured. Insert the USB stick. and press and hold the button for 5 seconds to open a dialogue. There the option “backup to USB” is available. Selecting it will save the settings and the device’s document.

## Restore backup

To reset the backup, make sure the same USB stick is connected to the i-See. Then, go to the menu item “documents”. And at the bottom of the screen, the USB icon is available. When this is selected, the device will recognise the backup file, and the same backup icon becomes available at the bottom of the screen. When this backup icon is selected, a list of available backup files is listed. In the name of the backup file, the date and time are present to distinguish several.

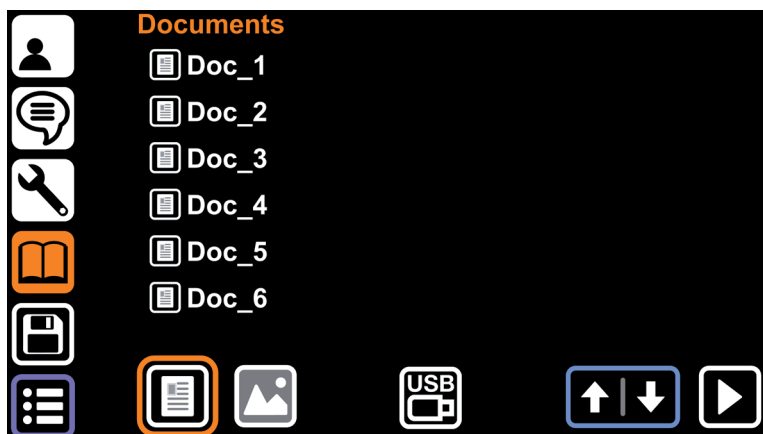
Hold the name of the file pressed for 2 seconds and press “restore device”. Here it is also possible to delete or change the name of the backup file.

## Documents

Apart from documents on the tableau, the i-See also has the ability to read other types of documents, like saved documents or PDF files, ePub or photos from a USB stick. In this Documents menu item, it is possible to select these.

This menu will display a list of all available documents on the internal memory. Use the arrow keys on the right side to scroll through the list. Underneath the list are buttons to filter the type of files by only documents, photos, or both.

Pressing the file name will select and open the document by the PLAY button at the right.

EN

## USB stick

A USB icon is displayed at the bottom of the screen when a USB stick is connected to the USB port on the left side of the device. Select this icon to switch to the USB storage instead of the internal memory. Now the device displays a list of all available documents from the USB stick.

## File menu

To delete, copy or rename files, keep the file name pressed until the file menu appears with the following options:

- Delete: Delete the selected file. Deleted files can not be recovered
- Rename: Change the document's name with the on-screen keyboard
- Export: Copy the file to a USB stick. Only available when USB stick is connected. Copying a scanned document from the i-See can only be displayed on other i-See devices. When you want to use it on another device, use export to TXT.
- Import: Copy the selected file from the USB stick to the internal storage.
- Exit: Close the File menu and return to the Documents tab.

## ePub

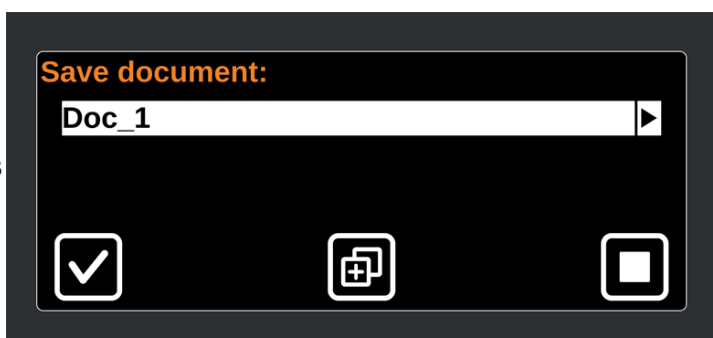
An ePub document can contain multimedia parts, like audio or video. Once these are available in the document, they will be displayed with an icon. Make sure the Zone detection is turned on, to select the multimedia items.

## Save

The i-See has the ability to save a scanned document. This can be done after the device has scanned and analysed a document for reading. Open the menu after this and go to the Save item.

A dialogue box opens to enter the file name. Press on the name to get the on-screen keyboard to change the name. Further, there are options to add extra pages to the document. The capture screen appears again to scan the next page. Press the button on the right to add another page, or press the checkmark on the left to go back to the document saving dialogue.

When ready, press the checkmark in the dialogue to save the document to the internal memory or press the stop button the cancel.

## 9 Troubleshoot

### General

| Problem description                                                                                                                                              | Actions                                                                                                                                                                                                                                                                                                                                                                                                                 |
|------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>The remote control is not responding.</p> <p><i>Please note: It can take around 10 seconds to create the wireless connection after pressing a button.</i></p> | <p>Is the remote control receiver (dongle) connected to the i-See at the back of the device?</p> <p>Is the battery empty of the remote control? It uses a single AAA battery.</p>                                                                                                                                                                                                                                       |
| <p>The device does not switch to external input.</p>                                                                                                             | <p>The i-See can serve as a monitor for a computer. Connect the HDMI cable between the PC and one of the two HDMI inputs at the back of the device.</p> <p>The recommended resolution should be 1920x1080.</p> <p>Press the power button until the source gets available or press the external input button on the remote control.</p> <p>If an X is displayed, check if the cable is well inserted into the ports.</p> |

### Live view mode

| Problem description                                     | Actions                                                                                                                                                                                                                                                                                                                                                                                                                      |
|---------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>The image or text is blurry and not in focus.</p>    | <p>Tap the magnifier icon (B2) to refocus. The focus is now blocked. Zoom in or out to activate the autofocus again.</p>                                                                                                                                                                                                                                                                                                     |
| <p>The text is not good in high-contrast mode.</p>      | <p>If the LED lighting has been changed, the quality of the high contrast mode may become more sensitive to noise or blurring. Increase the LED lighting accordingly.</p> <p>The contrast can be adjusted by swiping over the button (B3) from left to right, or right to left and holding your finger pressed until the letters become thicker.</p> <p>Make sure the function is activated in the “Live view settings”.</p> |
| <p>In the photo, the image is yellowish or blueish.</p> | <p>The white balance can be adjusted by swiping over the button (B3) from left to right or right to left and holding your finger pressed until the colors become more natural.</p> <p>Make sure the function is activated in the “Live view settings”.</p>                                                                                                                                                                   |
| <p>Part of the image is blanked.</p>                    | <p>The blanking reading aid is on. Press and hold the button (B3). Page blanking is switched off.</p>                                                                                                                                                                                                                                                                                                                        |

### Speech mode

| Problem description                                                           | Actions                                                                                                                                                                 |
|-------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>The i-See does not switch to speech mode by pressing the power button.</p> | <p>Wait until the egg timer has disappeared from the upper right corner.</p> <p>If a remote control is connected, flip the remote control to switch to Speech mode.</p> |

| Problem description                                                  | Actions                                                                                                                                                                                                                                                                                                                                 |
|----------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| During reading, the buttons at the bottom of the screen are missing. | The device is set to user interface preset 2. Tap at the lower left corner of the screen to let the buttons reappear.                                                                                                                                                                                                                   |
| The menu button is not available.                                    | The device is set to user interface preset 3. Go to the start screen ( with the yellow square) and press and hold the lower left corner of the screen until a message appears to open the menu. <i>This takes around 8 seconds</i> . Change in the System settings the interface preset to 1 or 2 in order to get the menu button back. |
| Missing colors in high contrast mode.                                | <p>In the menu, item System settings, you can make a selection which color combinations should be active. During use you can step through it by pressing button B1.</p> <p>The Live view mode and Speech mode have separate settings for the colors.</p>                                                                                |

Maintenance

Cleaning

For safety reasons, always disconnect the AC power adapter from the mains supply before cleaning the i-See.

Follow the instructions below to help prevent the product exterior from deteriorating or discolouration:

- Wipe with a soft, dry cloth.
- Do not use solvents or other chemicals. Do not wipe with a chemically-treated cleaning cloth.
- Use a soft dry cloth to clean all other accessories.

10 Recycling

The i-See Speech contains electrical and electronic equipment (WEEE) and should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will usually be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product and the packaging correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. If you are unsure, contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

Be aware of internal stored data/documents on the i-See Speech. Please delete these to protect your privacy.

# 11 Warranty

This product is covered by the warranty provisions of your supplier. For more information about these provisions, contact your supplier.

Never hesitate to contact your supplier or the manufacturer to report unexpected operation of incidents. Please note the serial number of the device when contacting the manufacturer.

EN

Manufacturer address  
production date

Catalog reference  
number

UDI number

Serial number

Device input power

Avoid listening to high volume  
levels for long periods

Complies with Part 15  
of the FCC Rules

**Rehan Electronics Ltd.,**  
Aughrim Road, Carnew,  
Co. Wicklow, Ireland

**2021-05**

**REF**

**2000-21-0016-22P-C3**  

**UDI**

**(01)05391513470684**  
**(21)AC00000**

**SN**

**AC00000**  
**DC in: 12V, 5A**

**MD**

**Electronic desktop  
magnifier**  
**Acuity Speech Matt F22P**

Medical device  
purpose and name

Read the instructions  
for use

Product conform  
EU standards

Indoor use  
temperature

IP rating

WEEE recycling

# 12 Technical specifications

|                                                                                                                        |                         |                                           |                |
|------------------------------------------------------------------------------------------------------------------------|-------------------------|-------------------------------------------|----------------|
| Resolution                                                                                                             |                         | 1920 × 1080                               |                |
| Video refresh rate                                                                                                     |                         | 50 Hz or 60 Hz                            |                |
| Display size                                                                                                           |                         | 54.6 cm<br>22 inch                        |                |
| Display modes<br><br><i>These display<br/>modes can be<br/>turned off in the<br/>i-See live mode<br/>advanced menu</i> | Photograph              | Full colour ( or high saturation)         |                |
|                                                                                                                        |                         | Greyscale                                 |                |
|                                                                                                                        | High contrast           | Black - white                             | Black - yellow |
|                                                                                                                        |                         | White - black                             | Yellow - blue  |
|                                                                                                                        |                         | Green - black                             | Blue - yellow  |
|                                                                                                                        |                         | Black - green                             | Red - black    |
|                                                                                                                        |                         | Yellow - black                            | Black - red    |
| Magnification<br>(With a 22" screen)                                                                                   | Minimum                 | ≈ 2.3× (when in crop mode ≈1×)            |                |
|                                                                                                                        | Maximum                 | ≈ 65×                                     |                |
| Reading Aid                                                                                                            | Horizontal Reading Line | Available in all models                   |                |
|                                                                                                                        | Vertical Reading Line   | Available in all models                   |                |
|                                                                                                                        | Horizontal Block Blank  | Available in all models                   |                |
|                                                                                                                        | Vertical Block Blank    | Available in all models                   |                |
| Audio performance                                                                                                      |                         | Mono 61mm speaker, 4W                     |                |
|                                                                                                                        |                         | Magnetic field: <0.5mT at 10cm distance   |                |
|                                                                                                                        |                         | Stereo on-ear headphone 32ohm, 3.5mm Jack |                |
|                                                                                                                        |                         | Magnetic field: <0.5mT at 3cm distance    |                |
| Preferred external HDMI input setting                                                                                  |                         | 1920x1080                                 |                |

|                                     |                                  |                                                                                                                                                                                                                      |                    |
|-------------------------------------|----------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------|
| Sound controls                      |                                  | Adjustable audio volume and reading speed                                                                                                                                                                            |                    |
| Supported languages                 | Belgian Dutch                    | Chilean Spanish                                                                                                                                                                                                      | Czech              |
|                                     | Belgian French                   | Argentinian Spanish                                                                                                                                                                                                  | Polish             |
|                                     | Danish                           | Colombian Spanish                                                                                                                                                                                                    | Russian            |
|                                     | Dutch                            | Mexican Spanish                                                                                                                                                                                                      | Hungarian          |
|                                     | US English                       | French                                                                                                                                                                                                               | Brazil. Portuguese |
|                                     | British English                  | Canadian French                                                                                                                                                                                                      | Greek              |
|                                     | Irish English                    | Italian                                                                                                                                                                                                              | Catalan            |
|                                     | Scottish English                 | Turkish                                                                                                                                                                                                              | Bulgarian          |
|                                     | Indian English                   | Norwegian                                                                                                                                                                                                            | Croatian           |
|                                     | South African English            | Swedish                                                                                                                                                                                                              | Romanian           |
|                                     | German                           | Finnish                                                                                                                                                                                                              | Slovak             |
|                                     | Spanish                          | Portuguese                                                                                                                                                                                                           |                    |
| Supported document file types       |                                  | VDF ( <i>i-See document format</i> ), PDF                                                                                                                                                                            |                    |
| Supported picture file types        |                                  | JPG, PNG, GIF, BMP, TIFF                                                                                                                                                                                             |                    |
| Internal storage                    |                                  | 1GB or 999 documents                                                                                                                                                                                                 |                    |
| Power Supply                        |                                  | Model: CGSW65-120-5000II<br>Input: 100-240VAC, 50/60Hz, 1.5A max.<br>Output: 12VDC, 5000mA max.                                                                                                                      |                    |
| Power rating                        |                                  | Standby: 0.2W<br>Maximum: 60W                                                                                                                                                                                        |                    |
| Dimensions                          | 22" (screen in highest position) | 48 × 48 × 65 cm<br>18.9 × 18.9 × 25.6 inch                                                                                                                                                                           |                    |
| Weight                              |                                  | 17 kg                                                                                                                                                                                                                |                    |
| Magnets                             |                                  | Speaker at the back of display:<br><div>&lt;0.5mT at 10cm distance</div> Upper corners of display:<br><div>&lt;0.5mT at 7cm distance</div> Remote control holder at tableau:<br><div>&lt;0.5mT at 4cm distance</div> |                    |
| IP rating                           |                                  | IP20; <i>solid foreign objects</i> ≥ 12.5mm<br><i>Not protected against ingress of water</i>                                                                                                                         |                    |
| Conditions of use                   | Temperature                      | 5 to 40 °C<br>41 to 104 °F                                                                                                                                                                                           |                    |
|                                     | Humidity                         | 0 to 75%, no condensation                                                                                                                                                                                            |                    |
| Conditions of storage and transport | Temperature                      | -25 to 70 °C<br>-13 to 158 °F                                                                                                                                                                                        |                    |
|                                     | Humidity                         | 0 to 90%, no condensation                                                                                                                                                                                            |                    |

**FCC NOTICE (Class B)**

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. this device may not cause harmful interference, and
2. this device must accept any interference received, including interference that may cause undesired operation. Federal Communications Commission Statement.

**EMI:**

The i-See Speech complies to harmonised standards according to medical devices and with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/television technician for help.

If you have any questions after reading this manual, please contact your supplier.

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