User manual
Bedienungsanleitung
Gebruiksaanwijzing
Manuel d' utilisation
Manuel del usuario
Manuale d'uso

Looky+ logo is a trademark of Rehan Electronics

First Edition (January 2010) V1.0


English	5	EN
Deutsch	17	DE
Nederlands	29	NL
Français	41	FR
Español	53	ES
Italiano	65	IT

1	Introduction.....	6
2	General description Looky+.....	6
3	Intended use	6
4	Maintenance.....	7
	4.1 Battery maintenance.....	7
5	Installation - batteries.....	8
6	Handle.....	8
7	Controls.....	8
	7.1 Switching the Looky+ on and off.....	8
	7.2 Digital zoom	9
	7.3 Selecting reading modes.....	9
	7.4 Freeze image.....	10
	7.5 Advanced settings.....	10
	7.5.1 Advanced settings - X button.....	10
	7.5.2 Advanced settings - Z button.....	11
8	Internal memory of the Looky+.....	11
9	Charging instructions.....	12
10	Troubleshoot.....	13
11	Specifications.....	14
12	Discarding the Looky+.....	15
13	Warranty regulations.....	15
14	Contact information supplier.....	15

EN 1 Introduction

Congratulations with the purchase of your Looky+! In this manual you will find a detailed description of all the functions of the Looky+. If you still have any queries after reading this manual, please contact your supplier (see the back of the manual for contact details).


2 General description Looky+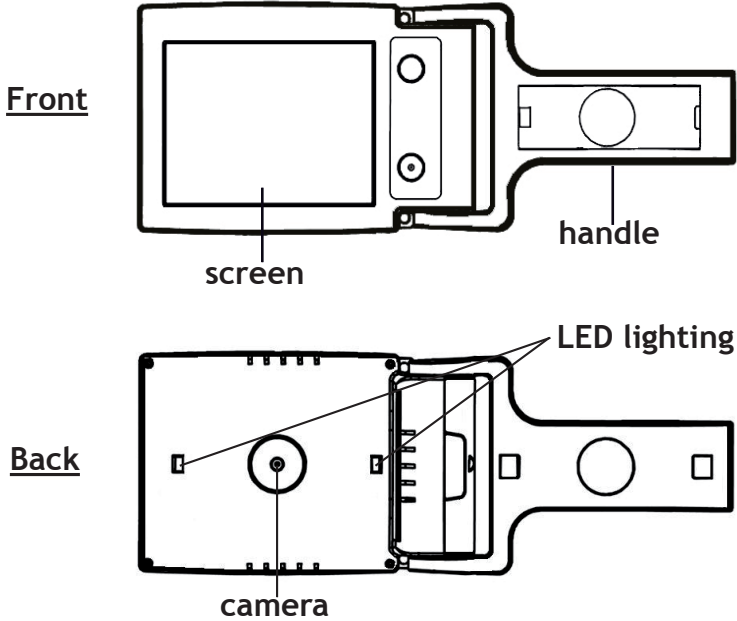

3 Intended use

This device is solely used for displaying enlarged representations of text, photographic material and objects. When the Looky+ is used for any purpose other than the intended use, the CE regulation will not apply.

Caution: do not, under any circumstances, perform maintenance or repairs to the device - always ask a producer approved technician to do this. Warranty will be voided if the Looky+ is attempted to be repaired by anyone other than a technician approved of by the producer.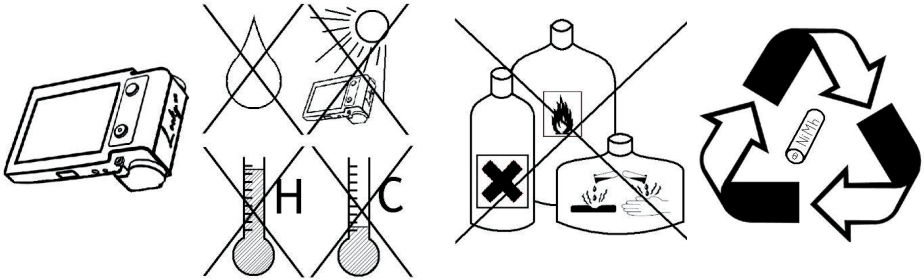

- Only use accessories for the Looky+ that are approved of by the producer.

4.1 Battery maintenance

- Remove the batteries from the Looky+ if you do not intend to use the device for an extended period of time. This will improve the longevity of the batteries.
- Batteries are sensitive to over-charging. It is recommended to only charge the batteries when they are completely drained.
- Chargeable NiMh batteries have an auto discharge that strongly depends on the temperature of the environment (e.g. never leave your batteries in a car that is parked in direct sunlight).
- New NiMh batteries must be charged several times before reaching their nominal capacity.
- Charge NiMh batteries at least once every 6 months.
- The average lifespan of a rechargeable battery is usually 300 charging cycles, but this depends on the use. Total capacity will slowly decrease after multiple charging cycles.
- Charge the batteries in a well ventilated environment.

5 Installation - batteries

1. Remove the lid from the battery compartment by pressing down on the arrow and moving the lid in the direction of the arrow.
2. Place the first battery at the back of the battery compartment.
3. Now place the second battery.

Note: charge the batteries before using your Looky+! See the charging instructions on page 12.

6 Handle

The handle of the Looky+ is closed on delivery. You can use the Looky+ with the closed handle by placing it directly on the text. When you extend the handle you can use the Looky+ like a magnifier. You now have the possibility to view objects or read texts.

It is also possible to use the Looky+ as an aid during writing. To do so you will need to extend the handle fully. When you now press the handle flat to the surface of the table, the screen will rise to an angle. The space underneath the Looky+ that is created in this way can be used to aid you during writing.


7 Controls

7.1 Switching the Looky+ on and off

- To switch the Looky+ on, keep button 1 pressed for 1 second. You will hear a low pitched tone followed by a high pitched tone. The screen is now active.
- To switch the Looky+ off, keep button 1 pressed until you hear a high pitched tone followed by a low pitched tone. The screen of the Looky+ will darken and the device will switch off.

7.2 Digital zoom

- Press the yellow button (button 2) to digitally enlarge the image on the screen.
- By pressing the yellow button (button 2) you can switch between the 4 zoom presets.
- By pressing the yellow button (button 2) after having selected the highest magnification, you will return to the lowest magnification.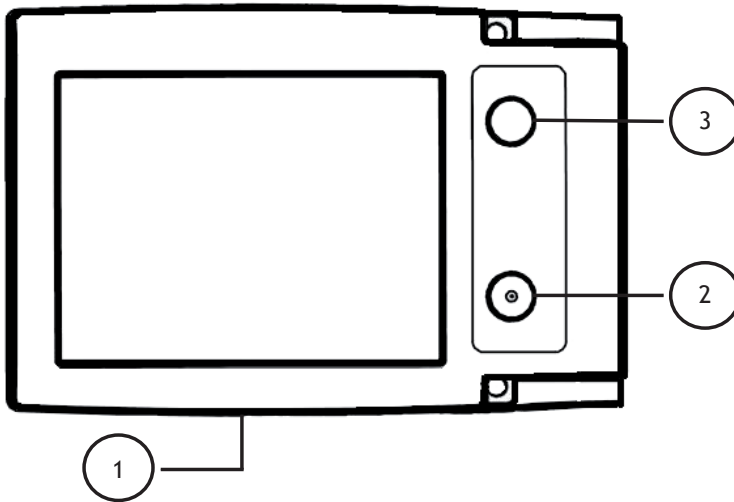

7.3 Selecting reading modes

- Press the blue button (button 3) to select different reading modes.
- By pressing the blue button (button 3) you will cycle through the following reading modes: Full Colour mode with LED lighting - White/Black - Black/White - Yellow/Blue - Yellow/Black.
- Keep the blue button (button 3) pressed for several seconds to switch to the full colour mode without LED lighting. When you now press the blue button (button 3) briefly, you will switch back to the previously used reading mode - the lights will also turn back on.

7.4 Freeze image

- By pressing button 1 briefly you can freeze the image on the screen.
- Press button 1 briefly again to unfreeze the image on the screen.
- When the image is frozen you can still cycle through the different reading modes by pressing the blue button (button 3). You can also still use the digital zoom function by pressing the yellow button (button 2).

Note: when the image is frozen, the LED lighting will be switched off in order to preserve energy.

7.5 Advanced settings

The Looky+ has 2 hidden buttons with which the advanced settings of the device can be operated. These buttons, named X and Z, are located in the battery compartment (the batteries must be removed from the Looky+ to access the X and Z buttons). Ensure that the adaptor is connected to the Looky+ and that the Looky+ is switched on to make use of the advanced settings.

7.5.1 Advanced settings - X button

With the X button you can turn the sound notifications of the Looky+ on or off. When pressing the X button, you will hear a sound that decreases in pitch - the sound notifications of the Looky+ are now switched off. To switch the sound notifications back on, press the X button again - you will hear a sound that increases in pitch.

Note: when the sound notifications have been switched off, the Looky+ will still give off an alarm sound when it detects a battery problem during the charging process. This sound is the only sound the Looky+ will produce when the sound notifications have been switched off.


7.5.2 Advanced settings - Z button

With the Z button you can adjust the brightness of the screen. By pressing the Z button you will lower the screen brightness by 20% - by doing so you will hear a sound that decreases in pitch. The lowest screen brightness is 20%, when you press the Z button once more the screen brightness will go back to 100% - when doing so you will hear a sound that increases in pitch. The Looky+ has 5 possible screen brightness settings: 100% - 80% - 60% - 40% - 20%.

8 Internal memory of the Looky+


The Looky+ stores the following settings in its memory when switched off and on:

- The reading mode that was used last (with exception of the full colour mode without lighting - instead of this mode, the Looky+ will start up in the reading mode that was used last before selecting the full colour mode without lighting).
- Whether the sound notifications are switched ON or OFF.
- The selected brightness of the screen.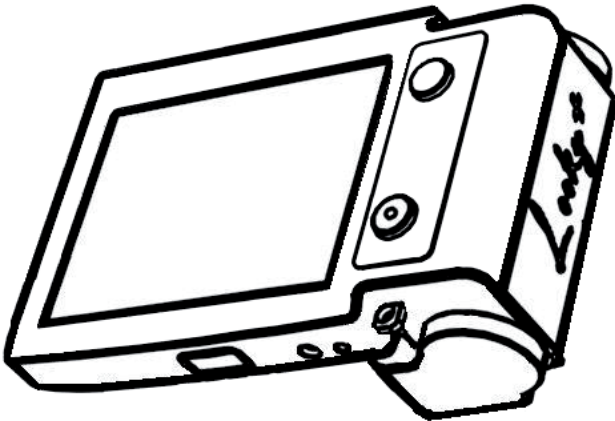

9 Charging instructions

Plug the adaptor into the wall socket and connect the plug into the power jack (1) of the Looky+. You will hear two brief beeps and the LED light (2) will be switched on during charging.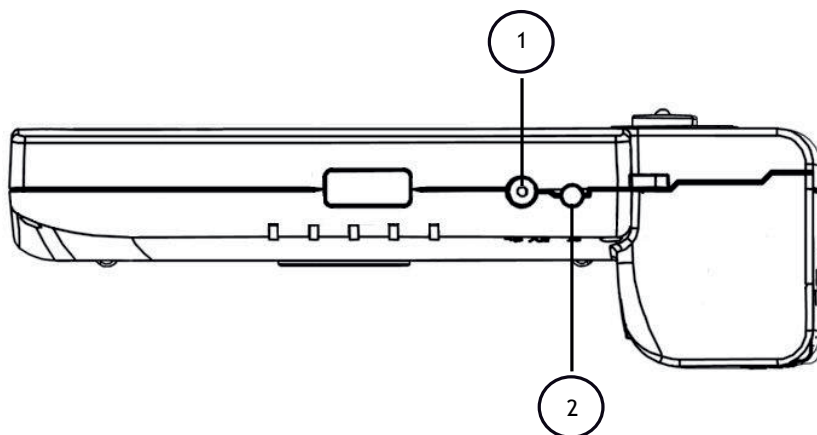

Charging time: ± 5 hours (depending on the battery capacity)

Time of use: ± 2 hours (depending on the battery capacity)

LED burning constantly	the batteries are being charged
LED blinking	the batteries are fully charged
LED blinking + audible dual signal	the Looky+ contains broken or non-rechargeable batteries

Note: the charging process will be interrupted when the Looky+ is switched on!

10 Troubleshoot

In this chapter you will find a description of some of the error messages the Looky+ can give, along with the corresponding solutions. Please read this section carefully if you are experiencing any problems with your Looky+. If you still have any queries after reading this section, please contact your supplier.

	Problem	Reason	Solution
1	The Looky+ switches itself off while sounding a tone that decreases in pitch	The batteries are too empty or too weak	<ul style="list-style-type: none"> - charge the batteries - use rechargeable batteries containing a capacity of $\geq 2300\text{mAh}$
2	The looky+ switches itself off while sounding a high pitched followed by a low pitched sound	The Looky+ will switch itself off if it does not detect movement within a 3 minute time span	<ul style="list-style-type: none"> - switch the Looky+ off if you do not use it - move the Looky+
3	The Looky+ cannot be switched on	The batteries are defect or an error has occurred in the Looky+	<ul style="list-style-type: none"> - replace the batteries - contact your supplier
4	The LED is blinking while the Looky+ is switched on and is not connected to the adaptor	An error has occurred in the Looky+	<ul style="list-style-type: none"> - contact your supplier
5	The Looky+ is sounding an alarm-like sound while charging	The batteries are defect or non-rechargeable	<ul style="list-style-type: none"> - replace the batteries: ensure that they are rechargeable and have a capacity of $\geq 2300\text{mAh}$

11 Specifications

Measurements (length x width x height) and weight (incl. batteries):

- Looky+ : 113 x 75 x 38 mm (\pm 200 gr.)

Screen:

- TFT diameter : 3.5 inch

Magnification:

- Minimal : \pm 2x
- Maximal : \pm 20x

Power supply:

- Batteries : 2x AA/LR6
- Adaptor : 5V DC 1,8A

Conditions of use:

- Temperature : $+5^{\circ}\text{C}$ til $+35^{\circ}\text{C}$
- Humidity : $<70\%$, no condensation

Storage and transport conditions:


- Temperature : $+0^{\circ}\text{C}$ til $+45^{\circ}\text{C}$
- Humidity : $<90\%$, no condensation

Time of use and charging time:

- You can use the Looky+ for \pm 2 hours at the recommended battery capacity of 2300 mA
- The average charging time of the Looky+ is \pm 5 hours at the recommended battery capacity of 2300 mAh.

12 Discarding the Looky+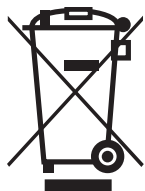

This symbol of the crossed out wheeled bin indicates that the product must not be disposed of with your other household waste. Check local regulations for proper disposal of electronic products.

13 Warranty regulations

This device is subject to the warranty regulations of your supplier. For further information regarding these regulations, please contact your supplier.

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Rehan Electronics Ltd.