

# LookyTAB 10



## Quick Start Guide

# Index

<b>1. English .....</b>	<b>3</b>
-------------------------	----------

# Safety Notice



The full user manual is loaded on the device and it is recommended that you read it. To open the user manual (Home screen>Applications>User Manual).

---



Caution: Avoid prolonged exposure to high volume levels, as it may impair your hearing.

---



Do not modify, open, or repair the device or its parts without the manufacturer's permission.

---



Temperature limits indicate the minimum and maximum permissible temperatures for storage, transport, or use.

---



Do not place the device or its parts in direct sunlight or near heat-transmitting objects.

---



Do not drop the LookyTAB 10 into any liquid, and refrain from using a wet cloth for cleaning.

---



Keep the battery and small parts (SD card, USB memory stick) out of reach of children.

---




Do not dispose of electronic and electrical equipment or accessories in household waste.

# Setting Up

## Opening device stand

Place the device with the screen facing upwards and the Zoom buttons on the right side.

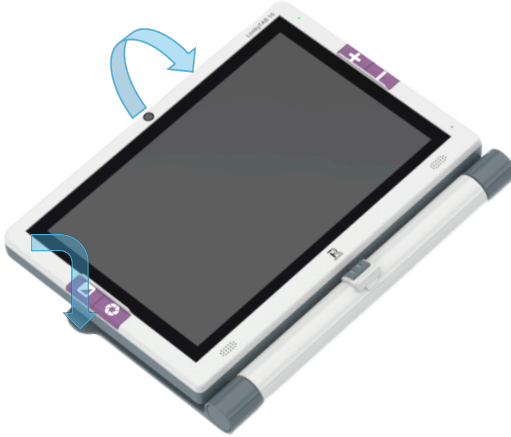


Hold the display with one hand at the top and hold the device stand with the other hand.



Raise the screen until it locks securely in the upright position, while keeping the stand in its table rest position.

## Closing device stand


Place one hand on top of the device and hold the device stand with the other hand.

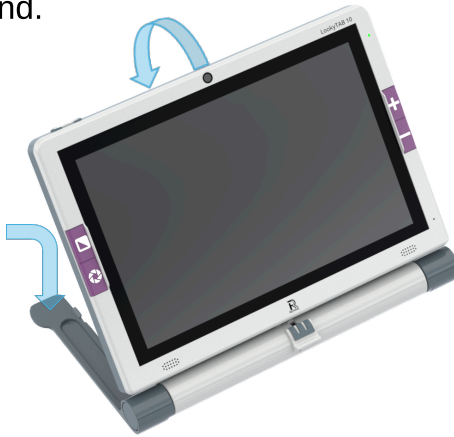

Push the screen down until it clicks and locks into the closed position.



## Slider and brake

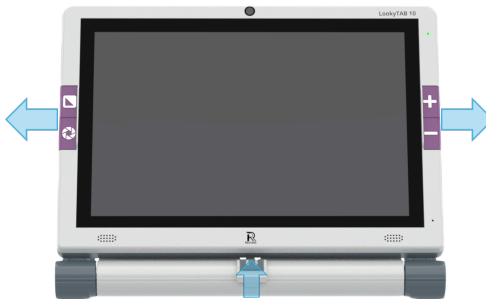

In the open position, drag the screen left or right to slide and read. Utilize the brake function to lock it in a horizontal position.

# Getting Started



- |                                   |   |
|-----------------------------------|---|
| <b>1. On / Off button:</b>        | Short press: Sleep/Resume.<br>3-second press: Device On/Off.      |
| <b>2. Zoom-In button:</b>         | Press button to increase magnification                            |
| <b>3. Zoom-Out button:</b>        | Press button to decrease magnification                            |
| <b>4. Contrast Colour button:</b> | Short press: Change Colour mode<br>Long press: All colour display |
| <b>5. Capture button:</b>         | Take snapshot of image  |



- 6. DC port:** Use this port to charge the LookyTAB 10 battery with the supplied DC charger. Make sure the battery is fully charged before first use.
- 
- 7. HDMI port:** Use this port to connect to an external display using the supplied HDMI cable.
- 
- 8. SD card slot:** Use this port to insert an SD card to store images, documents and videos.
- 
- 9. Micro-USB port:** This port is used for technical support only.
- 
- 10. USB-A port:** Use this port to connect an external USB stick or hard drive.
- 
- 11. Audio jack port:** Use this port to connect a headset.

**WARNING**

Only use certified cables and USB devices with the device. Avoid using non-certified cables and USB devices, as they may negatively impact the performance and functionality of the device.


# Home Screen



- |                                    |  |
|------------------------------------|--|
| <b>1. Magnifier button:</b>        | Tap to open the Magnifier application. |
| <b>2. Applications button:</b>     | Tap to open the Applications.          |
| <b>3. Favourite application:</b>   | Tap to add a favourite application.    |
| <b>4. Settings:</b>                | Tap to access Settings.                |
| <b>5. Wi-Fi On/Off button:</b>     | Long tap to open Wi-Fi settings.       |
| <b>6. Bluetooth On/Off button:</b> | Long tap to open Bluetooth settings.   |
| <b>7. Battery percentage:</b>      | Display the current battery status.    |
| <b>8. Date:</b>                    | Display the current date.              |
| <b>9. Time:</b>                    | Display the current time.              |



# Magnifier Application

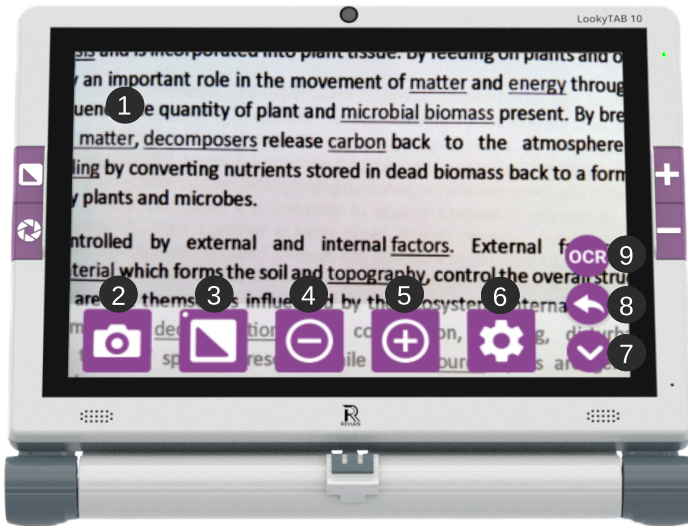

- |                                   |   |
|-----------------------------------|---|
| <b>1. Magnifier screen:</b>       | View a live preview of an object or document.                         |
| <b>2. Capture button:</b>         | Tap the Capture button to take a photo of your document.              |
| <b>3. Contrast colour button:</b> | Tap the Contrast Color button to apply and change the color contrast. |
| <b>4. Zoom out button:</b>        | Tap the Zoom Out button to decrease the magnification.                |
| <b>5. Zoom in button:</b>         | Tap the Zoom In button to increase the magnification.                 |
| <b>6. Magnifier settings:</b>     | Access the Magnifier settings to customize magnification options.     |
| <b>7. Hide menu:</b>              | Tap to hide the buttons banner.                                       |
| <b>8. Back button:</b>            | Tap to return to the home screen.                                     |
| <b>9. OCR button:</b>             | Tap to use OCR, (only available with OCR version of this device )     |

# Troubleshooting

## 1. Device fails to power on:

1. Connect the power adapter and charge the battery.
  2. If the battery is not charging, contact the local distributor or the manufacturer's service desk.
  3. If the power ON/OFF switch is not working, contact the local distributor or the manufacturer's service desk.
- 

## 2. Screen is blank:

1. If the device is in Standby Mode, press and release the power button.
  2. If the device powered off due to low battery, charge the device.
- 

## 3. Camera preview is not clear:

1. If the camera focus is unclear, select the correct focus mode in the magnifier settings.
  2. Adjust the distance between the camera and the document/object if there's not enough space for proper focus.
  3. In case of camera focus mechanism failure, contact the local distributor or the manufacturer's service desk.
- 

## 4. Speech audio is not audible:

1. If the system sound is OFF, turn ON system sound in the application settings.
  2. If speech mode is OFF, turn ON speech mode in the application settings.
  3. If the system volume is set too low, increase the device volume in the application settings.
- 

## 5. Smudges or blemishes on the screen:

1. Adjust brightness
  2. Clean camera and screen with soft cloth.
- 

## 6. For other issues:

1. Restore magnifier application settings from the menu.
2. If the problem persists, please contact your local distributor or manufacturer's service desk.





**Rehan Electronics Ltd.**

Brunswick Row  
Aughrim Road  
Carnew  
County Wicklow  
Republic of Ireland

**Phone:** +353 (0) 53 942 6742

**Fax:** +353 (0) 53 942 6051

[sales@rehanelectronics.ie](mailto:sales@rehanelectronics.ie)  
[technical@rehanelectronics.ie](mailto:technical@rehanelectronics.ie)  
[www.rehanelectronics.com](http://www.rehanelectronics.com)